



## User information for stainless steel wire

- Dust and fume may be generated during processing e.g. in welding, cutting and grinding.
- Use a dust suction device in the laboratory.
- Do not expose to high vibrations to avoid hardening of the wire.
- Safety glasses are recommended.
- The durability of the wire is unlimited.

## Disinfection and cleaning:

The products are supplied in non-sterile packaging.

## Storage instructions:

Dry

Normally the products have to be protected for chemicals, acids and extreme temperature fluctuations. Do not expose to high vibrations.

## Please note the symbols on the packaging:



Single use  
Lot-number - for traceability of the respective production batch  
Order-number  
Observe the instructions for use